BREMER HALO CROWN[®] TRACTION SET (HT025, HT026, HT025HI, HT026HI)

Application, Removal and Post Removal Pin Site Care Instructions

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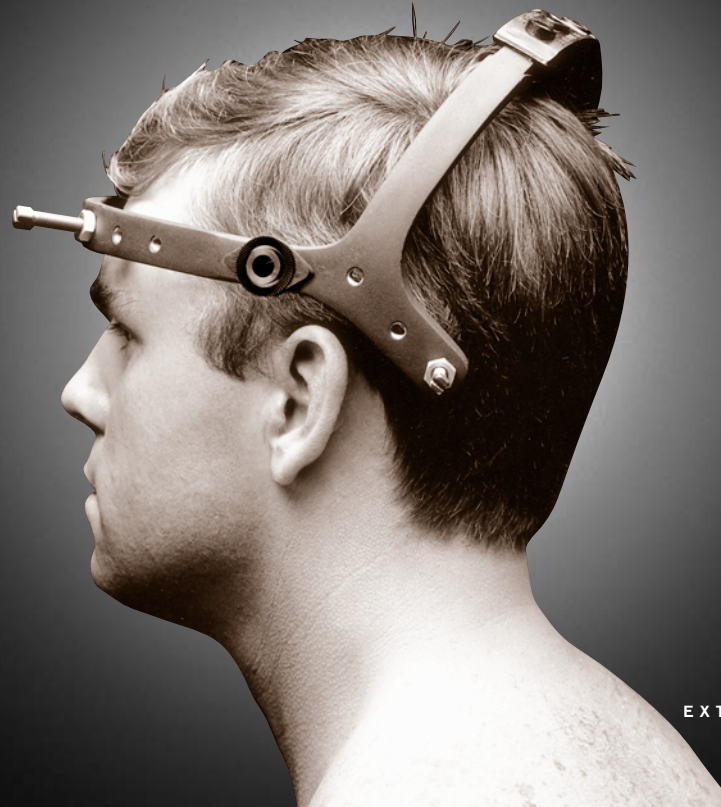
PRELIMINARY APPLICATION NOTES

The patented Bremer Halo Crown® and Torque Limiting Caps are engineered to permit the added safety of application without moving the patient. The Halo Crown applies easily to the supine patient while the head remains on a firm surface such as a rigid spinal board or the hard pad of an Emergency Room examination table.

MRI Compatibility: Bremer crowns and rings have been tested and found MRI compatible in standard MRI systems to 1.5 tesla. Use of high field systems, upgraded systems, and variations in equipment and software, including increased scan frequency, placing more than two pins posteriorly or putting the coil too close to the crown or

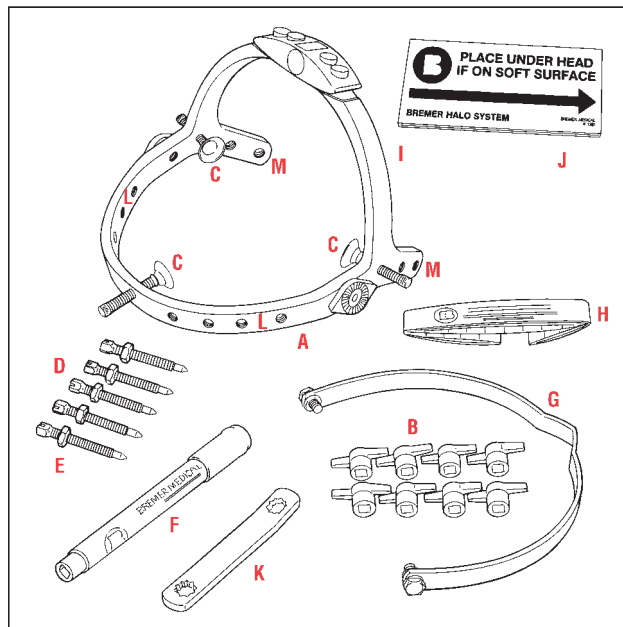
ring, can influence MRI compatibility. Check with your MRI manufacturer's technical service department for suggested protocol.

Note: If the surface is soft, it may be necessary to insert the enclosed corrugated head pad beneath the patient's head before beginning application to optimize head position relative to the patient's body.

Important: Instructions in this guide are for application of the Halo Crown on adults with good bone quality. If this product is to be applied to patients with compromised bone quality or pediatric patients, it is the responsibility of the surgeon to review patient status and literature to ensure proper pin torque and

crown positioning is used. If skull fracture is suspected a preapplication CT scan is suggested to ensure skull pins are not placed in a fracture site.

Important: The patented Bremer Torque Limiting Caps are preset to break off at 8in/lbs. of torque, simplifying crown application and eliminating the need for an application torque driver in most cases. **Torque Limiting Caps are not for use on pediatric patients or for patients with compromised bone quality.** For these applications use of a torque driver set to a torque level appropriate for patient condition should be used.

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*Adult Use only

‡Optimal pin site may vary according to patient condition. Choose most appropriate site.

Supplies Needed During Application

- Syringe
- Thin, 1-1/2" Needle
- Local Anesthetic
- Sterile Gloves
- Surgical Prep
- 4"x 4" Gauze Pads
- Scissors
- Prep Razor
- Narrow (#11) 1-1/2" Scalpel

Additional items (discretionary purchase)
Additional Skull pins, pack of five, sterile with 8 torque limiting caps

- HiFix™ pins AC020
- Standard pins AC014
- Adjustable torque driver AC002
- Locking Traction Bail HT003S

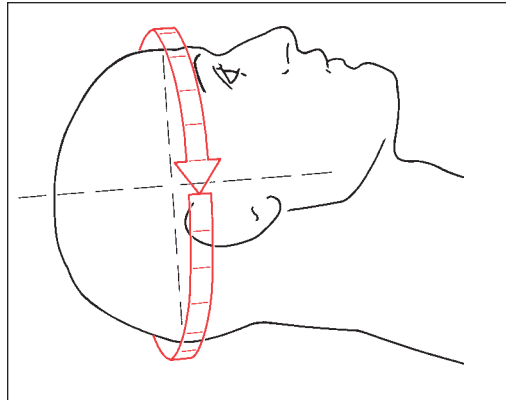
BEFORE APPLYING THE BREMER HALO CROWN®

Measure The Patient

Measure the circumference of the skull at the equator.

Caution: Be certain tape is pulled tight to assure measurement does not include hair.

For infants or toddlers a pediatric ring (Item #PT001 or PT002) is suggested. For larger patients or patients with abnormal anatomy a custom ring (Item #HT014) is suggested. To order a custom ring contact your local sales representative or DePuy Spine customer service (800-227-6633) for measuring protocol.

Select The Appropriate Halo Crown

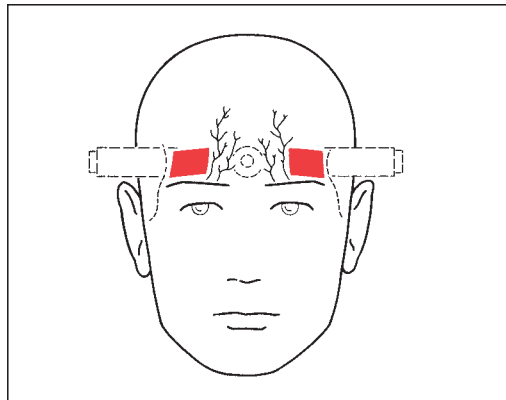
- Small 19" to 23" (48-58cm)
- Large..... 23" to 26" (58-66cm)

Note: Use smallest possible size.


Prep The Pin Sites

Prepare the pin sites using surgical prep; areas may be shaved.

Optimal anterior sites are just above the lateral 1/2 of the eyebrow. Care should be taken to avoid the supraorbital nerves.

Anterior Pin Sites Safe Zone.

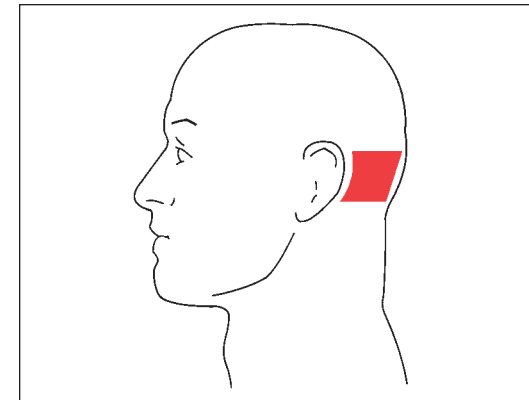
Posterior Pin Sites Safe Zone.

(Adapted from Garfin,SR, Botte MJ, Waters RL, et al. Complications in the Use of the Halo Fixation Device. J Bone Joint Surg Am. 1986; 68-A: 320-325.)

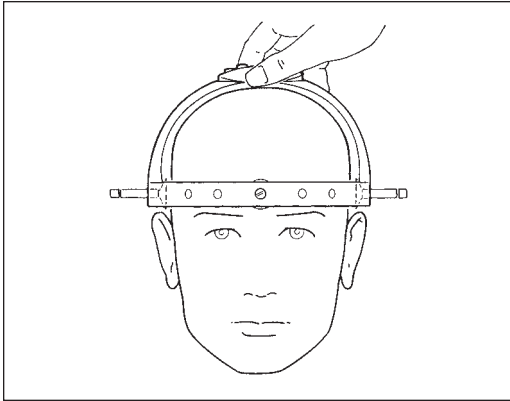
APPLICATION OF THE BREMER HALO CROWN®


1 Place Halo Crown on patient's head.

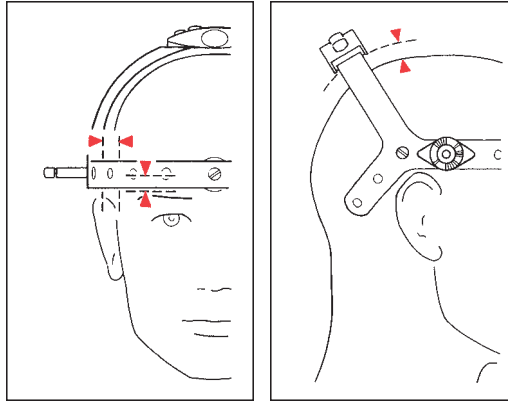3 Snug positioning pads to maintain Halo Crown position and assure correct alignment.

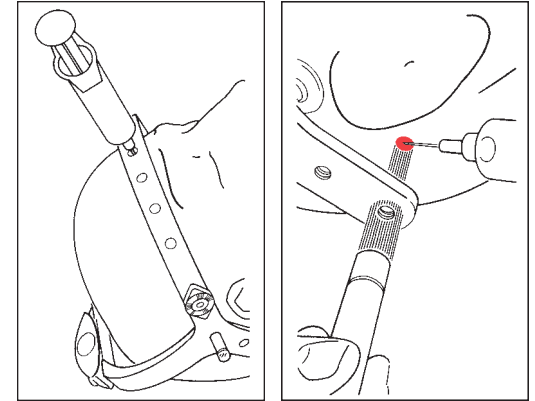- 1cm at the pin sites
- 1cm above eyebrows
- Not touching ears
- Posterior pin sites below equator of skull
- Capital arch not touching top of head


4 Place the needle through the chosen pin holes and anesthetize the periosteum and skin.

Caution: When anesthetizing anterior pin sites, have patient close eyes.


Note: Use a pen light or a sterile swab handle, directed through the pin holes, to help locate pin sites for administration of local anesthetic.

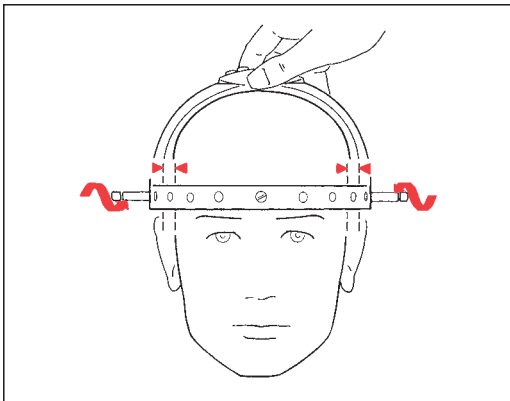Note: It is suggested that a narrow (#11) scalpel be used to make an incision down to the level of the periosteum at the insertion point of the skull pin.

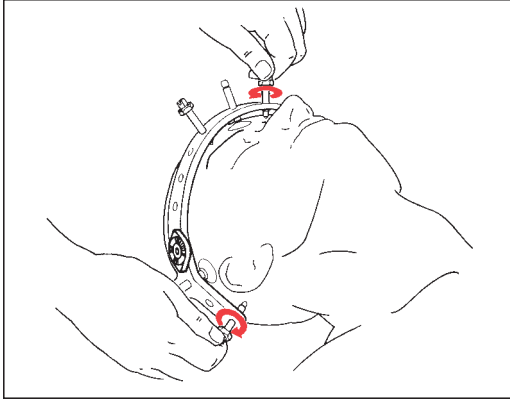
2 Adjust positioning pads and Halo Crown to ensure 1cm separation between the Crown and head at the pin sites.


- 5** With patient's eyes closed and while maintaining Halo Crown position, insert skull pins in selected holes tightening opposing pins with the fingers until penetrating the skin.

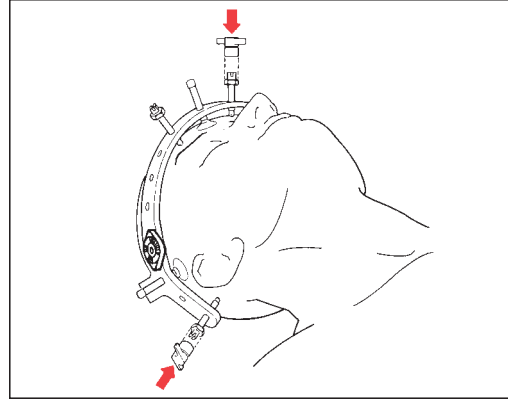Important: If Halo Crown becomes misaligned during this step, back skull pins away from the head, reposition the Crown and return to step 5.


- 6** Place torque limiting caps* on each of the four pins and simultaneously tighten opposing pins, slowly, two turns at a time.

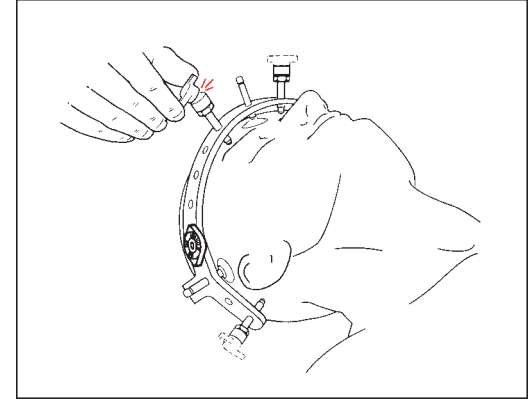*Not for pediatric use.

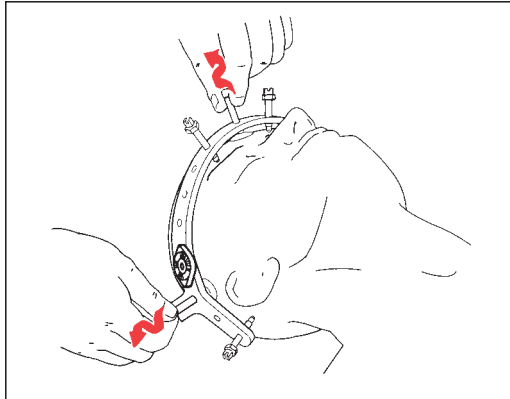- 7** Continue to alternate between opposing pin sites, tightening two turns each time, until torque limiting caps break off. Make sure lock nuts are up near head of pin as shown.

8 Remove all three positioning pads and pins and discard.

9 Patient may now be placed in a Bremer AirFlo® Vest, Bremer Classic II Vest or traction hoop attachment as indicated.

Note: For proper application technique of an AirFlo Vest or Classic II Vest see instructions enclosed in the procedure set with each product.

Caution: Before applying traction or moving a patient on whom an AirFlo® or Classic II Vest has been applied, or after approximately 15 minutes, re-torque the skull pins using the four remaining torque limiting caps following steps 6 and 7 – then firmly secure the locking nuts on all four pins against the Halo Crown with the 7/16" (11mm) wrench supplied.

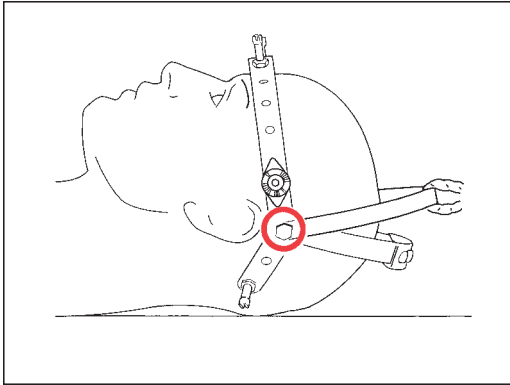
Protocol For Re-torquing Skull Pins

Caution: Before checking pin torque loosen lock nuts with 7/16" wrench and re-tighten before patient is moved. A thin coin may be used to hold the pin in place during lock nut loosening or tightening.


- 1** Pins should be re-torqued using an appropriate torque pin driver according to the following protocol:
- 2 24 to 48** hours after initial application: 8 in/lbs.
- 3** Every 2 to 3 weeks or on patient complaint of pain at pin sites:
2 to 3 in/lbs.

APPLYING TRACTION

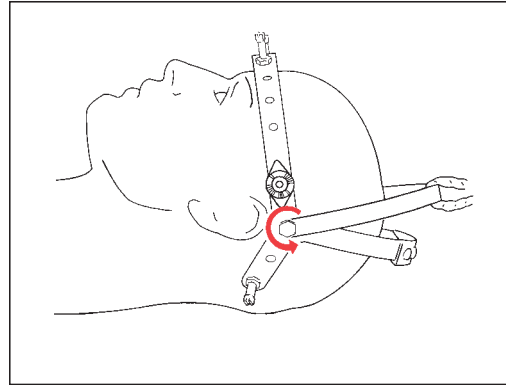
- 1 Apply the traction hoop attachment by placing the bolts in the holes at the juncture of the Halo Crown and capital arch.
- 2 Apply traction according to physician's instructions.

- 3 To adjust flexion/extension as indicated by physician, maintain the patient's head position while slightly loosening the traction hoop attachment bolts.

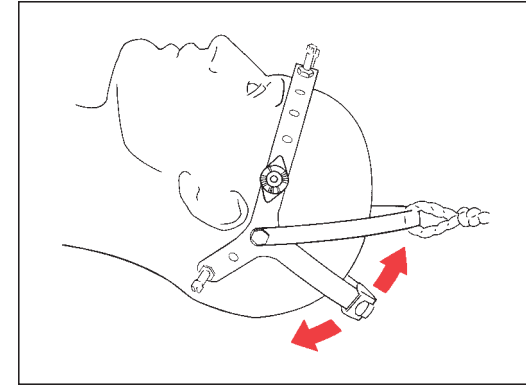Important: Due to variations in bone densities and application techniques, traction weight is at the discretion of the physician. Traction is advised not to exceed 50 lbs. If greater traction is anticipated the Trippi-Wells Tong (product #1373) or Universal Tong (product #1374) should be used.

- Note:** A disposable traction bail is included with every crown kit. For control of flexion and extension during traction, use the adult locking traction bail (product #HT003S).
- 4 Physician can now rotate patient's head to desired position and retighten bolts.

REMOVING THE BREMER HALO CROWN®

Note: Safe removal of a Bremer Halo System requires 2 healthcare professionals. A cervical collar may be applied prior to removal of vest and crown.


Supplies Needed

- | | |
|----------------------------|---------------------------------------|
| Gauze 4" x 4"s | (1) 7/16" (11mm) |
| Hydrogen Peroxide Solution | Wrench |
| (2) Adhesive Bandages | (1) Thin coin to use as a screwdriver |
| | (1) Cervical Collar |


Note: All skull pins should be removed from the Crown or Ring and discarded in an approved sharps disposal container.

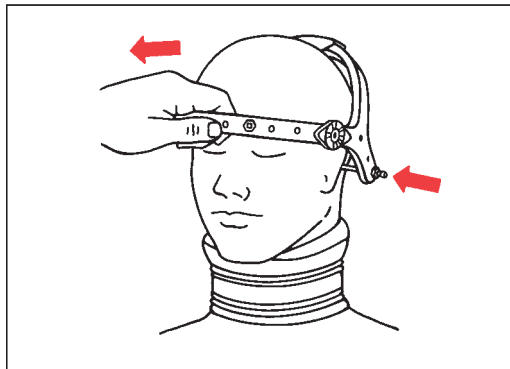
Warning: Using a screwdriver for this procedure is contraindicated and may result in serious injury to the patient.

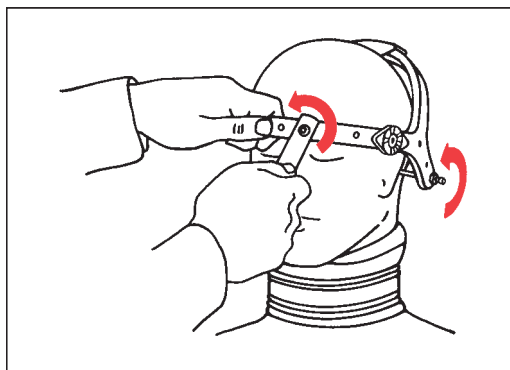
1 The assistant should grasp the Crown above the patient's nose applying slight anterior pressure holding the points of the posterior pins firmly against the skull.

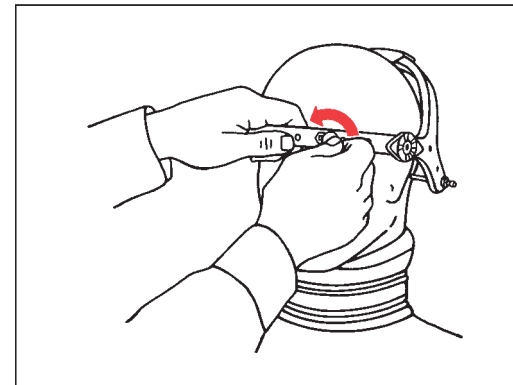2 Loosen all of the locking nuts on the skull pins using the 7/16" (11mm) wrench.

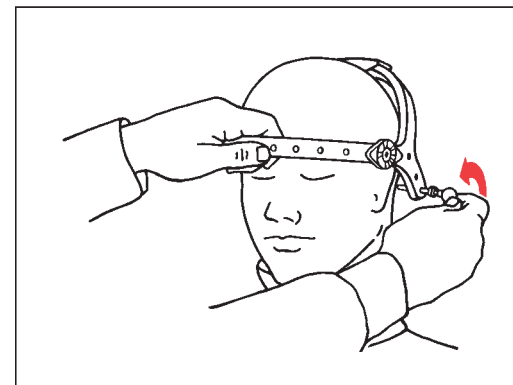3 Using a coin, remove the anterior skull pins.


4 While the assistant maintains anterior pressure on the Crown, remove one posterior skull pin using a coin.

Note: Trimming of the hair away from the posterior pins is often necessary to prevent tangling.


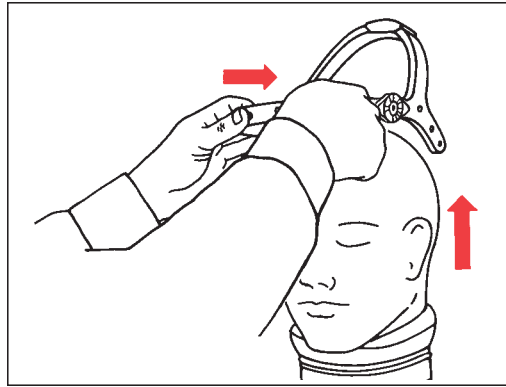
- 5 The assistant can now remove the Crown by applying posterior pressure until the last skull pin comes away from the head and the Crown can be lifted off.

CARE OF PIN SITES

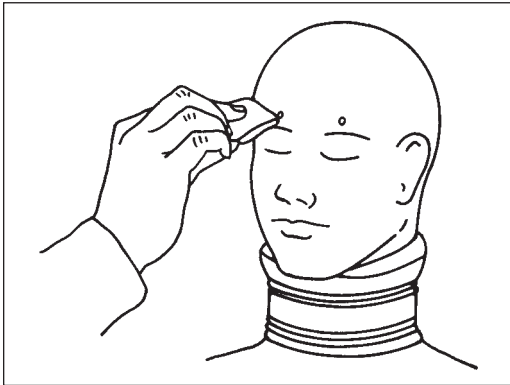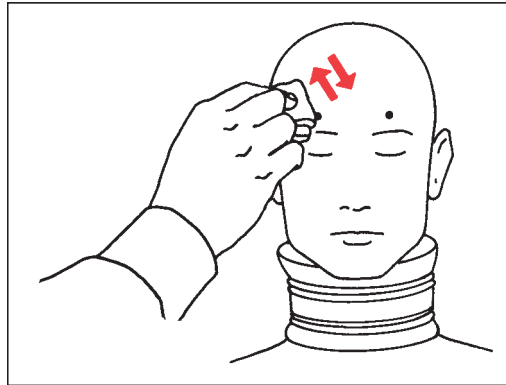
- 1 The pin sites should be thoroughly cleaned using hydrogen peroxide solution and gauze 4"x4"s.

- 2 To reduce scarring, a hydrogen peroxide saturated gauze 4"x4" can be used to massage the anterior pin sites to break the adhesions formed between the periosteum and the skin.

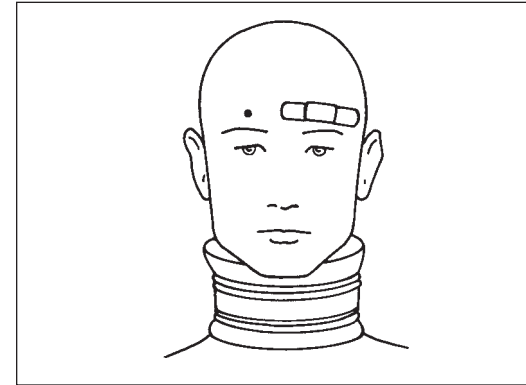Note: This can be quite painful and sometimes results in some bleeding. Patients should continue to move the skin over the pin sites for several days to avoid reattachment of the adhesions.

- 3 Adhesive bandages should be applied over the anterior pin sites for several hours until they are tightly closed.

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Authorized European Representative:
DePuy International LTD, St. Anthonys Road,
Beeston, Leeds LS11 8DT, ENGLAND

Limited Warranty: DePuy Spine standard catalog products are sold with a limited warranty to the original purchaser against defects in workmanship and materials; any other express or implied warranties, including warranties of merchantability or fitness, are hereby disclaimed. Please see the current price list for important warranty information.

®Indicates a registered trademark of DePuy Spine, Inc.
The following U.S. Patent apply: 4,612,930, 4,838,264, 5,121,741.
© Copyright DePuy Spine, Inc., 2003. All rights reserved. Printed in the USA.

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SYMBOL TRANSLATION			
LOT NUMBER	CATALOG NUMBER	QUANTITY	MATERIAL
LOT	REF	QTY	AcroFlex/Titanium = AT Aluminum = A Aluminum/Plastic = AP Foam = F Nickel/Titanium = NT Plastic = PL Plastic/Foam = PF Polyester = PE Polymer = P Polymer/Carbon Fiber Composite = PC
SINGLE USE	SEE INSTRUCTIONS FOR USE	USE BY	Stainless Steel = S Stainless Steel/Aluminum = SA Stainless Steel/Phenolic = SP Stainless Steel/Utem = SU Titanium and its Alloys = T
STERILIZATION BY IRRADIATION	STERILIZATION BY ETHYLENE OXIDE		
STERILE	NON-STERILE		

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© Copyright DePuy Spine, Inc. Alle rechten voorbehouden. Gedrukt in de VS.
© Copyright DePuy Spine, Inc., 2003. Tous droits réservés. Imprimé aux Etats-Unis.
©Copyright DePuy Spine, Inc., 2003. Alle Rechte vorbehalten. Gedrukt in den USA.
© Copyright Bremer Medical, Inc. 2003. Tutti i diritti riservati. Stampato negli U.S.A.
© Copyright DePuy Spine, Inc., 2003. Todos los derechos reservados.
© Copyright DePuy Spine, Inc., 2003. Med ensamrätt. Tryckt i USA.